

# How to Apply the VRTX CTO and VRTX CTO4



CTO

CTO4


1



## PLACEMENT OF BACK PANELS

With the head in neutral position, center the back thoracic panel and back collar panel. Ensure the back of the collar panel is between the ears and upper shoulders.


2

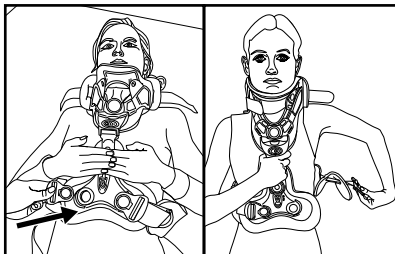

## PLACEMENT OF FRONT PANELS

Center the front thoracic panel and front of the collar.

3

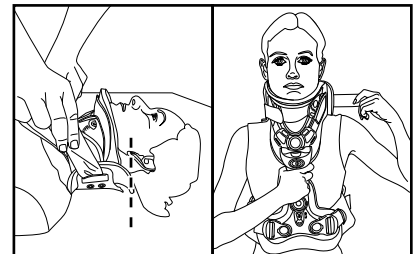

## TIGHTENING THE STRAPS

While holding the front panel, attach the loop straps on each side. To tighten, peel back the loop straps one at a time, tighten and re-attach.


4



## CLIPPING IN THE STRAPS

While holding the front panel, clip in both thoracic straps.


5

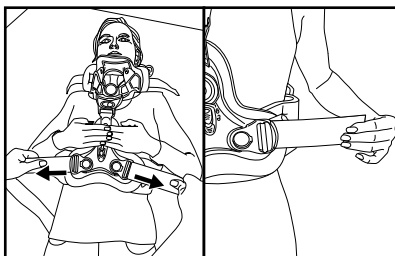

## TIGHTENING THE STRAPS

Pull the thoracic straps to tighten ensuring the brace is snug.

6

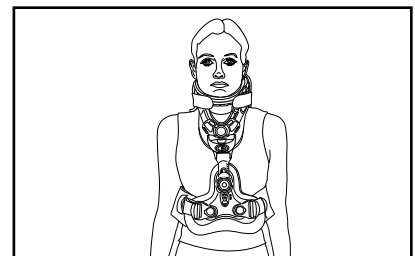

## CHECK THE FIT

Ensure both the chest panel and collar are snug.

