

How to Apply the

VRTX Cervical Collar VRTX TX Collar VRTX MultiPost Collar



Cervical




TX



MultiPost


1

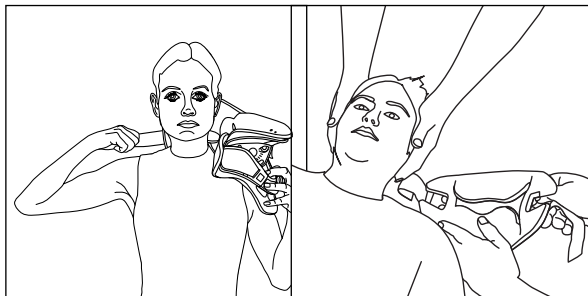

PLACEMENT OF BACK PANEL

With the head in neutral position, place the back panel so that it is centered on the head and between the ear and upper back.


2

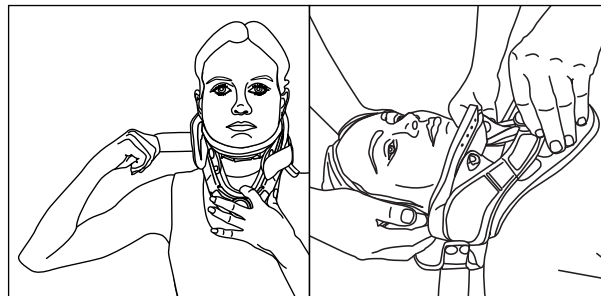

PLACEMENT OF FRONT PANEL

Position the front panel under the chin with the sides up and over the collar bones.


3

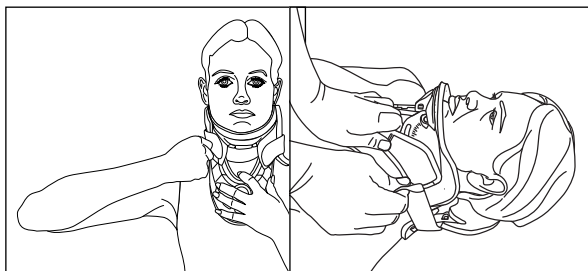

TIGHTENING THE STRAPS

While holding the front panel, attach the loop straps on each side. To tighten, peel back the loop straps one at a time, tighten and re-attach.

4

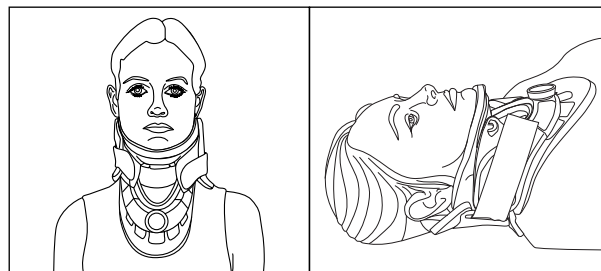

COLLAR ON CORRECTLY

Ensure both straps are firmly in place and that the collar is snug.

✓

CHECK THE FIT

- 1. Chin Dots Hidden?** The chin piece plastic should be flush with the end of the chin, hiding the "chin-dots" indicator on the chin pad.
- 2. Height Correct?** Collar height is correct when it supports your head in neutral alignment.
- 3. Collar Snug?** Evenly tighten and re-tighten the straps for a secure fit.
- 4. Airway Clear?** Check that the back of the chin piece does not contact the throat. If it does, release the strap and reposition the chin piece. Recheck to ensure the airway is clear.

