

HORIZON PRO 627 LUMBAR, 631 LSO, 637 LSO PATIENT APPLICATION

PRODUCT INFORMATION

HORIZON PRO 627 LUMBAR, 631 LSO, 637 LSO




HORIZON PRO 627 LUMBAR



HORIZON PRO 631 LSO




HORIZON PRO 637 LSO

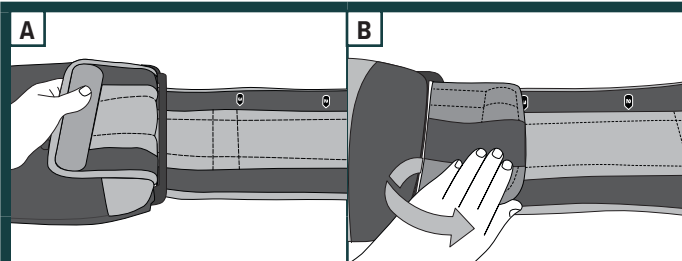

SIZE ADJUSTMENT


1A. To change the size of the Horizon Pro, peel up the inside end of the side panel on each side. Adjust to proper fit and reattach. 1B. The Horizon Pro Brace sizing follows closely to standard pants sizing. Size ranges are only guidelines.

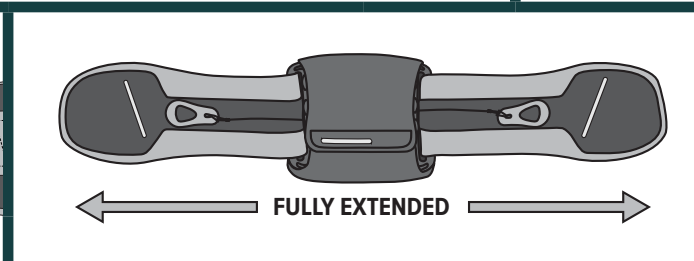

FULLY EXTENDED


APPLYING THE BRACE

2. Place the pull-tabs on the belt 2 inches (5 cm) from the plastic in the back of the Horizon Pro. Pull the ends of the belt away from each other ensuring that the belt is fully extended.

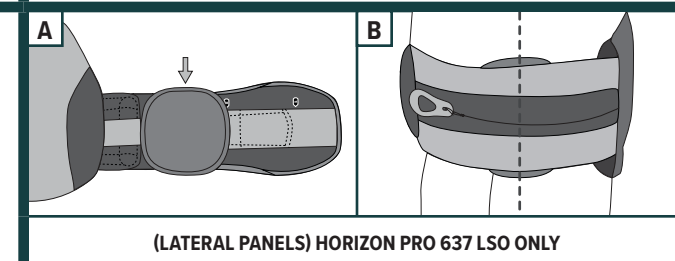


(LATERAL PANELS) HORIZON PRO 637 LSO ONLY


3A. With brace extended fully and the inside facing up, place one lateral panel on the midline on each side panel of the brace. Press down to secure to brace. 3B. When Horizon Pro brace is properly fit to the patient, the lateral panels should fit at patient's midline.

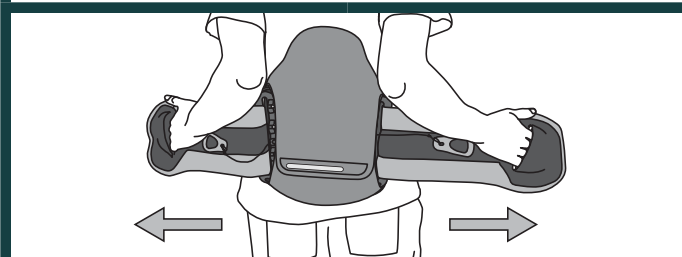

4. Place the back panel so that the bottom of the back panel is at or near the top of the buttocks.

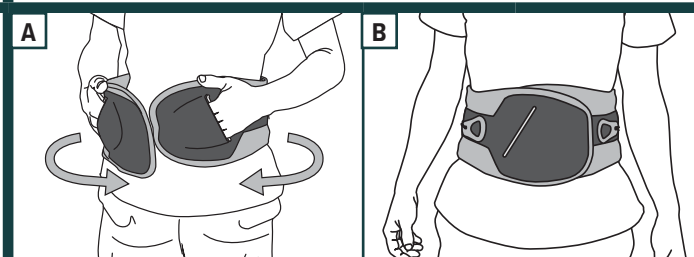


5A. Apply the brace tightly around the abdomen, the right side overlapping the left. 5B. When applied properly, both the upper and lower edge of the brace should be snug. If there is any gap, reapply the brace and angle the ends to insure a snug fit at both the upper and lower edge.

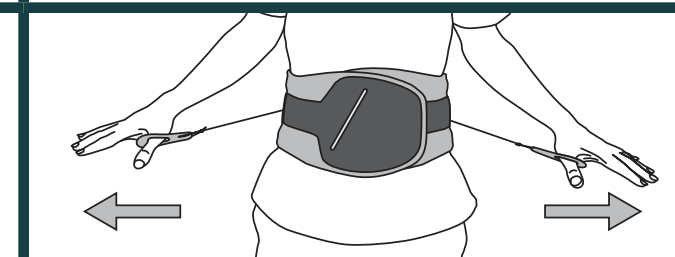


TIGHTENING

6. Slip your thumbs through the holes in the pull tabs. Pull away from your body.

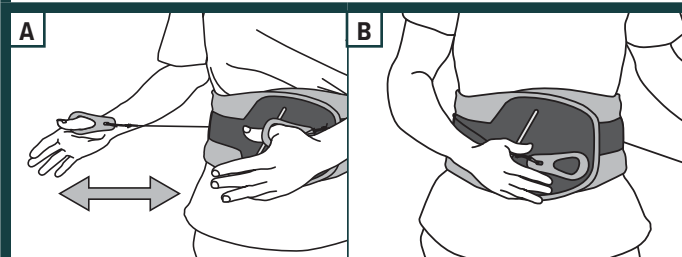


7A. The left pull tab controls lower compression and the right pull tab controls the upper compression. Adjust the compression as needed. 7B. When tightened, secure the pull tabs to the hook-and-loop in front of the brace.

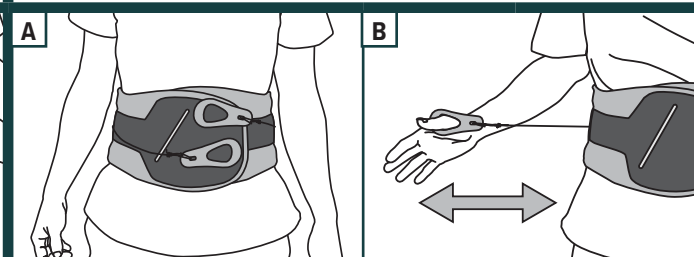

ADJUSTMENT

8A. Adjust the upper and/or lower compression by loosening or tightening the pull tabs as needed. 8B. When patient is seated, slightly loosening the upper portion may be beneficial.

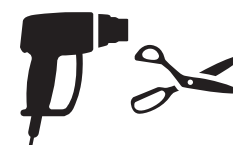

THIS PRODUCT MAY BE SUBSTANTIALLY MODIFIED BY AN INDIVIDUAL WITH EXPERTISE

For more information, please visit www.aspenmp.com/education/product-modifications